



Via Cristans, 24  
33085 Maniago PN

Tel: +39 0427 739132

www.defcon5italy.com  
info@defcon5italy.com

P.Iva: IT-01568980930

## TACTICAL ACCESSORIES

### DEFCON 5 TOURNIQUET IN BLACK COLOR

**CODE: YL-8412**

**BRAND: DEFCON 5 2024/2025**

The **CAT TYPE tourniquet** is the CE-certified benchmark for firefighters, police officers, and first responders. Designed to quickly stop severe bleeding, it can be applied **one-handed**, thanks to its military-origin design.

Essential for managing **catastrophic hemorrhage (C / M / X)**, it is a vital tool that should always be immediately accessible.

#### Main Features

- Highly effective in stopping arterial bleeding
- Usable on arms and thighs
- **CE standard** tourniquet
- Rod pressure resistance: **470 N**
- Rod deformation resistance: **17.7 mm**
- Compression rod with **dual-cone grip** for secure handling
- **Yellow strap tip** for high visibility, even in low light
- Strap material: **8-strand braided Nylon®**
- Operates in extreme temperatures: **-20°C to 80°C**

#### General Use Instructions (catalog summary)

1. Place the tourniquet between the wound and the heart.
2. Pull the strap tight and secure it.
3. Twist the compression rod until bleeding stops.
4. Lock the rod in the holding system.
5. Secure the free strap.
6. Close the white tab and **record time and date**.

*For leg hemorrhage: apply high on the thigh and tighten using both hands.*

### **Precautions**

- **Single use** – dispose after application
- Use only by trained personnel
- Never loosen an applied tourniquet
- Dispose as medical waste
- Store between **5°C and 30°C** in a dry, ventilated area
- Shelf life: **5 years**

### **Dimensions**

- Open length: **66 cm**
- Folded length: **15.5 cm**
- Width: **4 cm**
- Folded thickness: **3 cm**

Weight: 97 grams

**WEIGHT (KG):** 0.10

## GALLERY

